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UV protection level 2-1.2 according to DIN EN 170

### Use:

The watch glass bandage is intended for single use only. **To ensure optimal adhesion, the skin should be clean, free of grease and dry.** Bushy eyebrows can affect the adhesion of the patch: Shave or pluck the eyebrows if necessary.

Please remove the cover paper carefully. Place the watch glass bandage centrally on the eye so that the break-proof protective cover is centered over your eye and your eye is not covered by the plaster. (OKKLUglas AERO EXTRA UV: The large adhesive surface faces up to the eyebrow.) Fix the patch to the skin with gentle pressure.

Please consult your doctor about the medication.

### Intended purpose:

Ready-to-use eye bandage with transparent, break-proof protective cover: hypoallergenic, ergonomic and extremely skin-friendly

### Indication:

To be applied as mechanical and uv protection of the eye after ptosis operations, for lagophthalmos, and after eye surgeries.

### Contraindications:

Allergy to plasters / plaster intolerance.

### Effect:

Mechanical barrier feature; provides optimal air circulation through ventilation openings; UV protection class 2-1.2 according to DIN EN 170 through UV-stabilized plastic component

### Warnings:

**Spatial vision will be severely restricted when wearing a watch glass bandage (roadworthiness). The plaster must not be adhered to injured skin or mucous membranes.**

Serious incidents occurring in connection with the product must be reported to us as the manufacturer and/or the competent authority of your member state.

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#### Berenbrinker Service GmbH