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### OKKLUEASY

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### OKKLUEASY

**Intended purpose:**

Soft-Fleece occlusion eye patch for the attachment to glasses

**Indication:**

Amblyopia therapy of eyeglass wearers. Alternative product in case of plaster intolerance and otherwise insufficient therapy compliance

**Contraindications:**

No known contraindications

**Effect:**

Covering and opaque darkening of the eye with better eyesight so that the weak eye will be “forced” to function, and will be trained (occlusion therapy).

**Warnings:**

**Spatial vision will be severely restricted when wearing OKKLUEASY. The medical device should be worn only in case of an appropriate indication or prescription from a doctor or orthoptist.**

**Target patient population:**

Children, application supported by their parents. Please observe the occluding instructions provided by your ophthalmologist / orthoptist!

**Application:**

Slip the OKKLUEASY over the temple with the opening facing toward the Velcro side. Velcro strip 1 should always be at the bottom. Then pull the OKKLUEASY further along the temple so that it completely covers the lens. Close the Velcro behind the nose pad.

**CAUTION:** If the glasses have no nose pads, make sure that the Velcro strip is closed between the glasses and the face.

In some cases it is helpful to shape the OKKLUEASY to achieve a perfect fit. Also please take care that the Velcro strips always lie on top of one another to guarantee proper fit of the OKKLUEASY. Please always follow the occluding instructions provided by your ophthalmologist / orthoptist!

**Washing instruction:**

Hand wash; not tumble dryable. Tip: If you dry OKKLUEASY with a hairdryer at low heat, the fleece fabric will become „fluffy“ again.

You are also welcome to watch the video for fixing OKKLUEASY on our website at [www.berenbrinker.de](http://www.berenbrinker.de).

Serious incidents occurring in connection with the product must be reported to us as the manufacturer and/or the competent authority of your member state.

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**Berenbrinker Service GmbH**