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### OKKLUEASY

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### OKKLUEASY

**Intended purpose:**

Soft-Fleece occlusion eye patch for the attachment to glasses

**Indication:**

Amblyopia therapy of eyeglass wearers. Alternative product in case of plaster intolerance and otherwise insufficient therapy compliance

**Contraindications:**

No known contraindications

**Effect:**

Covering and opaque darkening of the eye with better eyesight so that the weak eye will be “forced” to function, and will be trained (occlusion therapy).

**Warnings:**

**Spatial vision will be severely restricted when wearing OKKLUEASY. The medical device should be worn only in case of an appropriate indication or prescription from a doctor or orthoptist.**

**Target patient population:**

Children, application supported by their parents. Please observe the occluding instructions provided by your ophthalmologist / orthoptist!

**Application:**

First pull of the top single-sided Velcro strap to uncover the holes provided for the temples. Then detach the double-sided Velcro strap from the bottom tape. Then grasp the right temple – if the right eye is to be occluded – or the left temple – if the left eye is to be occluded – and carefully push it through the visible, larger hole, and pull the fabric across the lens as far as the nose bridge (caution: Velcro fastener should be facing upwards). Then re-attach the double-sided Velcro strap to the lens, with the Velcro strap facing upwards. Finally, grasping the remaining single-sided Velcro strap, pass it under the bridge of the nose inwards and attach it to the top of the double-sided Velcro strap. Please make sure that the Velcro fasteners are always properly positioned on top of each other so that the eye patch fits correctly, does not slip, and also looks good.

**Washing instruction:**

Hand wash; not tumble dryable. Tip: If you dry OKKLUEASY with a hairdryer at low heat, the fleece fabric will become „fluffy“ again.

You are also welcome to watch the video for fixing OKKLUEASY on our website at [www.berenbrinker.de](http://www.berenbrinker.de).

Serious incidents occurring in connection with the product must be reported to us as the manufacturer and/or the competent authority of your member state.

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**Berenbrinker Service GmbH**