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OKKLUglas AERO UV and OKKLUglas AERO EXTRA UV

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Anwendung:

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UV protection level 2-1.2 according to DIN EN 170

Use:

⊗ The watch glass bandage (AERO UV variant: 74 mm x 93 mm x 8,5 mm / AERO EXTRA UV variant: 90 mm x 93 mm x 8,5 mm) is intended for single use only.

☂ Store dry

To ensure optimal adhesion, the skin should be clean, free of grease and dry. Bushy eyebrows can affect the adhesion of the patch: Shave or pluck the eyebrows if necessary.

Please remove the cover paper carefully. Place the watch glass bandage centrally on the eye so that the break-proof protective cover is centered over your eye and your eye is not covered by the plaster. (OKKLUglas AERO EXTRA UV: The large adhesive surface faces up to the eyebrow.) Fix the patch to the skin with gentle pressure.

Regarding the medication and the optimal variant, please consult your doctor.

Intended purpose:

Ready-to-use eye bandage with transparent, break-proof protective cover: hypoallergenic, ergonomic and extremely skin-friendly

Indication:

To be applied as mechanical and uv protection of the eye after ptosis operations, for lagophthalmos, and after eye surgeries.

Contraindications:

Allergy to plasters / plaster intolerance.

Effect:

Mechanical barrier feature; provides optimal air circulation through ventilation openings; UV protection class 2-1.2 according to DIN EN 170 through UV-stabilized plastic component

Warnings:

Spatial vision will be severely restricted when wearing a watch glass bandage (roadworthiness). The plaster must not be adhered to injured skin or mucous membranes.

Serious incidents occurring in connection with the product must be reported to us as the manufacturer and/or the competent authority of your member state.

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**Berenbrinker Service GmbH**